**THOMSON**

*Wireless headphones / Supplement headphones*

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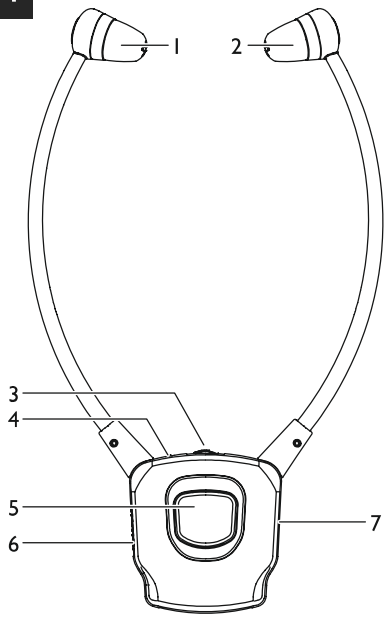
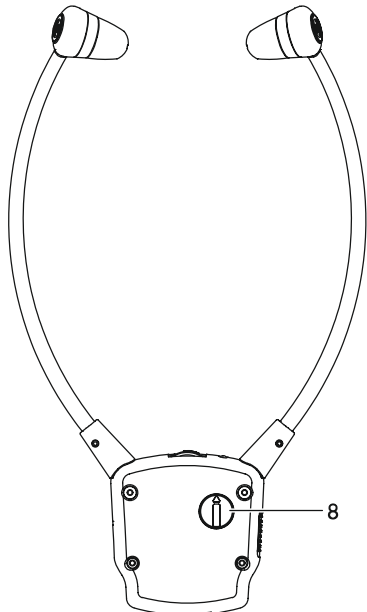
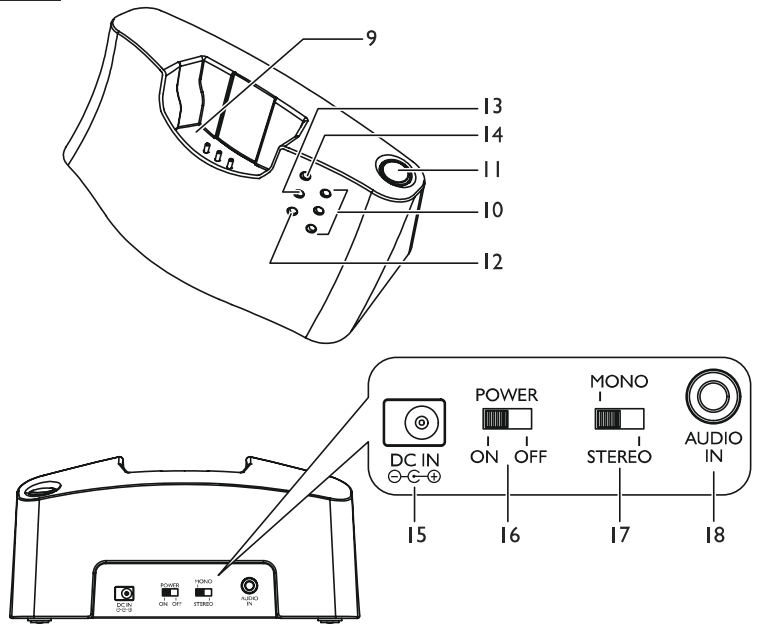
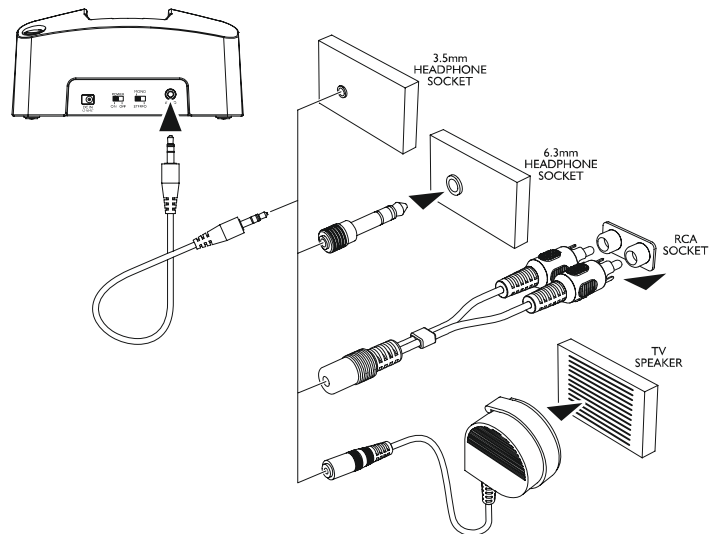
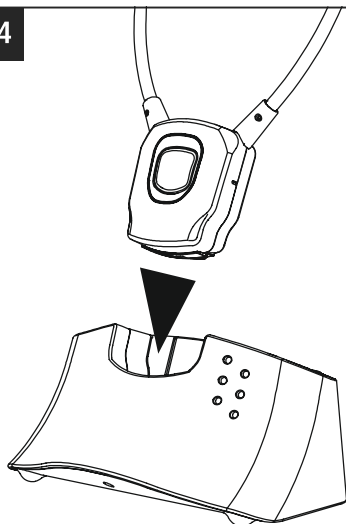
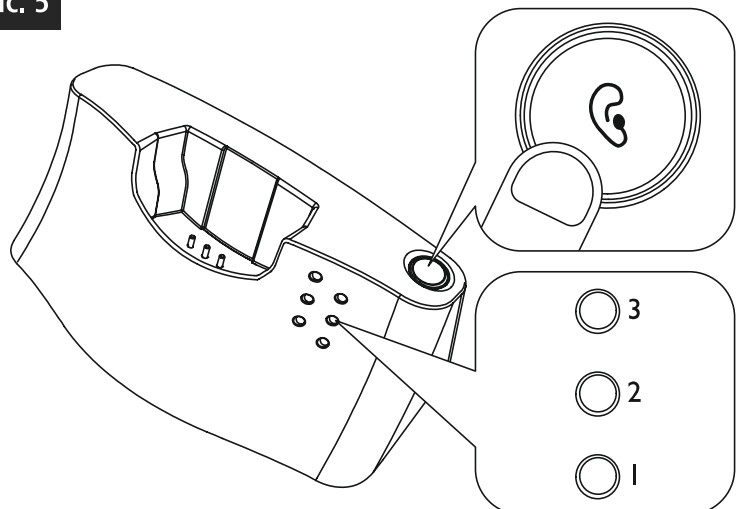
**Pic. 4**

**Pic. 5**

Thank you for choosing a Thomson product. Take your time and read the following instructions and information completely. Please keep these instructions in a safe place for future reference. If you sell the device, please pass these operating instructions on to the new owner.

## 1. Explanation of Warning Symbols and Notes

### Warning

This symbol is used to indicate safety instructions or to draw your attention to specific hazards and risks.

### Note

This symbol is used to indicate additional information or important notes.

## 2. Overview

### Receiver - Fig. 1

1. Adjustable earphone right
2. Adjustable earphone left
3. Volume control
4. LED power/signal display
5. Switch for internal microphone
6. Power switch on/off
7. Integrated microphone
8. Left/right balance adjustment

### Transmitter - Fig. 2

9. Charging cradle
10. Display of current sound/speech clarity settings (1, 2 or 3)
  - 3 = amplified bass
  - 2 = amplified treble
  - 1 = amplified bass and treble
11. Button for adjusting speech clarity
12. LED power (on/off) display
13. LED audio signal display
14. LED to display battery charge
15. Socket for power supply
16. Power switch on/off
17. Mono/stereo switch
18. 3.5 mm audio-in socket

## 3. Package contents

### 3.1 Item WHP5305BK/00131970, 1T131970

- Headset receiver with built-in lithium polymer rechargeable battery
- Transmitter and charging station
- Plug-in power supply
- Cinch adapter cable – 2x cinch to 3.5 mm mini stereo jack
- Audio cable (3.5 mm mini stereo jacks on both ends)

- 6.3 mm to 3.5 mm stereo adapter plug
- External microphone
- 1 pair of spare ear cushions (smaller size)

### 3.2 Item A305/00131971

- Headset receiver with built-in lithium polymer rechargeable battery
- 2 pairs of silicone earbuds

## 4. Safety Notes

### Warning

- This product was designed specifically for persons with impaired hearing. Therefore, it is capable of producing unusually high volumes.
- Before using this product, persons with normal hearing should turn the volume of the playback device down to the lowest level!
- This device operates in the 2400 MHz band. Since this product may interfere with the operation of pacemakers, persons with pacemakers should consult a doctor before purchasing this product.
- Using this product limits your perception of ambient noise. For this reason, do not operate any vehicles or machines while using this product.
- Always keep the volume at a reasonable level. Loud volumes, even over short periods, can cause hearing loss.

- Never open, damage or swallow batteries or allow them to enter the environment. They can contain toxic, environmentally unfriendly heavy metals.
- Do not tamper with or damage/heat/disassemble the batteries/rechargeable batteries.
- The product is intended for private, non-commercial use only.
- Use the product for its intended purpose only.
- Children should be supervised to ensure that they do not play with the device.
- Do not use the product in areas where the use of electronic devices is not permitted.
- Connect the product only to a socket that has been approved for the device. The electrical socket must always be easily accessible.
- Disconnect the product from the network using the power button – if this is not available, unplug the power cord from the socket.
- Lay all cables so that they do not pose a tripping hazard.
- Do not bend or crush the cable.
- Do not drop the product and do not expose it to any major shocks.
- Dispose of packaging material immediately according to locally applicable regulations.

- Do not modify the product in any way. Doing so voids the warranty.
- Do not use the product if the adapter, connection cable or power cable is damaged.
- Do not attempt to service or repair the product yourself. Leave any and all service work to qualified experts.
- Do not open the device or continue to operate it if it becomes damaged.

## 5. Introduction

This 2.4 GHz digital wireless chin headset operates using the latest radio technology and allows even older people to enjoy clean, crystal-clear sound when watching TV and listening to music at home.

Simply connect the transmitter with the desired audio source: a DVD player, CD player, TV set, hi-fi system or iPod. You'll be enjoying vivid, mind-blowing sound in minutes – no complicated wiring or installation required. And the built-in external microphone will make sure that the entire family can enjoy watching TV together at a comfortable volume for everyone.

### Note

Always remember to reduce the volume when switching to the internal microphone. Otherwise the volume may increase suddenly. This can cause hearing damage.

## 6. Data

- 2.4 GHz digital frequency jumping technology
- Charging station with LED display
- Automatic power-off of the HF transmitter (when no audio signal is being received)
- Mute switch for speech capture with the internal microphone
- Quick-charging system
- Microphone adapter for direct audio capture from the speakers of the playback device (e.g. TV, radio, etc.)
- Left/right balance adjustment
- 4 selectable sound settings
  - Neutral
  - Amplified bass and treble
  - Amplified treble
  - Amplified bass
- Mono/stereo switch
- Volume control

## 7. Technical specifications

<b>Transmitting mode</b>	Digital frequency jumping technology
<b>Carrier frequency</b>	2,4 GHz
<b>Operating voltage</b>	Transmitter: 5 V, 550 mA Receiver: 3.7 V, 350 mAh lithium polymer rechargeable battery
<b>Frequency range</b>	30 Hz – 20 KHz
<b>Distortion</b>	< 0,5 %
<b>Signal-to-noise ratio</b>	> 75 dB
<b>Range</b>	Up to 20 metres (depending on the composition of intervening walls)

## 8. Transmitter installation, fig. 3

- Connect the power supply to the power socket on the rear side of the AV selector.
- Connect the power supply to a mains socket.
- The cinch adapter cable, audio cable, 6.3 mm-to-3.5 mm adapter plug and an external microphone offer a wide variety of ways to connect to your audio source (such as a TV, hi-fi system, mp3 player or iPod).
- Put the power on/off switch on the rear side of the transmitter in the ON position.

### Note

- Depending on your TV, using its 3.5 mm headphone jack may mute the sound through the speakers. Switching to the microphone means that the headphones will reproduce the sound only after it has been emitted by the TV's speakers. This will allow everyone in the room to hear the TV at a comfortable volume, including people other than the person wearing the headphones.
- The external microphone will also pick up sounds from the surroundings. This will allow you to hear the doorbell or telephone ringing even while using the headphones.

The red power LED will light up as soon as the power supply is connected to the mains. When an audio signal is received from the audio source, the green LED will shine steadily.

## 9. Getting Started

### 9.1 Connecting headphones to the transmitter

- Turn on the audio source connected to the transmitter (TV or audio player). The green LED will now shine steadily.
- Put the on/off switch of the headset receiver in the ON position. Now the green signal display LED will light up.
- Set the volume to the desired level.
- When the battery's charge runs low, the green signal display LED on the headset will begin to flash.

#### Note

Before using the headphones for the first time, make sure that the battery is fully charged. Charge the battery for at least 3 hours before the first use. Make sure that the headphones are switched off during charging.

The transmitter will automatically switch off if it receives no audio signal for approx. 5 minutes, or if the signal from the audio source is too weak for flawless reception. As soon as the signal strength is sufficient, the transmitter will automatically switch back on.

### 9.2 Connecting two headphones to the transmitter (requires additional headphones (00131971) in combination with the base set (00131970, 1T131970))

- Switch the power switch to OFF on both pairs of headphones and the transmitter.
- Switch both pairs of headphones to ON. The green LEDs will flash.
- Press and hold the MIC buttons on the headphones for about 5 seconds. The green LEDs on the headphones should flash rapidly. Now switch the transmitter to ON as well.
- The headphones and the transmitter will now be automatically coupled, and the green LEDs on the headphones will remain lit.

#### Note

Both headphones will automatically reconnect to the transmitter even after being switched off.

## 10. Charging, fig. 4

- Put the on/off switch of the headphones in the OFF position.
- Place the headphones on the transmitter station and make sure that the charging contacts are touching the charging pins.
- Charging begins automatically. The LED display will light up steady red. When the LED changes from red to green, the battery is 80% charged. The headphones must remain on the charging station for at least an hour to fully charge.

#### Note

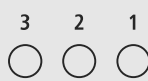
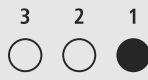
- To prevent damage to the batteries, always switch off the headphones when they are not in use.
- Before using them for the first time, the batteries should be fully charged. To guarantee an acceptable charge, the batteries should initially be charged for 3 hours. Make sure that the headset receiver is switched off during charging.

## 11. Speech/sound clarity, fig. 5

If necessary, the headset receiver's speech/sound clarity can be optimized and adjusted from 1 to 3 using the orange LEDs on the transmitter.

Simply press the pictured button to adjust the speech/sound clarity.

The speech (and sound) clarity can be improved by making the following adjustments on the transmitter:

Setting	Orange LED display	Effect
Standard		Neutral
1		Amplified bass and treble
2		Amplified treble
3		Amplified bass

## 12. Troubleshooting

### 12.1 No sound

- Check that the power supply plug is fully inserted into the mains socket and the other end is properly connected to the power supply socket on the transmitter.
- Make sure that the power on/off switches on both the transmitter and the headset receiver are in the ON position.
- The headset receiver's battery may have insufficient remaining charge. Charge the battery.
- Check that your TV, hi-fi system or audio source is switched on and a channel is correctly configured to send an audio signal.
- Increase the volume using the control on the headset receiver.
- The connected audio/video device may not be in playback mode. Please start the playback.
- The output volume of the audio source may be too low. Set the volume.

## 12.2 Distortion

- The battery in the headset receiver may have insufficient charge. Charge the battery.
- The distance between the headset receiver and the transmitter may be too great. Move the two devices closer together and turn the headset receiver in the direction of the transmitter.
- The input level of the audio signal may be too low. Increase the volume of the audio source.

### 12.3 If the headset receiver cannot be connected to the transmitter, proceed as follows:

- Put the on/off switch of the headphones in the OFF position.
- Switch the headset receiver to ON.
- Press and hold the MIC button for approx. 5 seconds. The green LED on the headset receiver should begin to flash quickly. Now switch the transmitter to ON as well.
- The headset receiver and the transmitter will not be automatically paired, and the green LED on the headset receiver will be lit steady green.

## 13. Care and Maintenance

Only clean this product with a slightly damp, lint-free cloth and do not use aggressive cleaning agents.

## 14. Warranty Disclaimer

Hama GmbH & Co. KG assumes no liability and provides no warranty for damage resulting from improper installation/mounting, improper use of the product or from failure to observe the operating instructions and/or safety notes.

## 15. Service and Support

Please contact Hama Product Consulting if you have any questions about this product.

Hotline: +49 9091 502-115 (German/English)

Further support information can be found here:

[www.hama.com](http://www.hama.com)

## 16. Recycling Information

### Note on environmental protection:

After the implementation of the European Directive 2012/19/EU and 2006/66/EU in the national legal system, the following applies:

Electric and electronic devices as well as batteries must not be disposed of with household waste. Consumers are obliged by law to return electrical and electronic devices as well as batteries at the end of their service lives to the public collecting points set up for this purpose or point of sale. Details to this are defined by the national law of the respective country.

This symbol on the product, the instruction manual or the package indicates that a product is subject to these regulations.

By recycling, reusing the materials or other forms of utilising old devices/Batteries, you are making an important contribution to protecting our environment.

## 17. Declaration of Conformity

**CE** Hereby, Hama GmbH & Co KG declares that the radio equipment type [00131970, 1T131970] is in compliance with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at the following internet address:

[www.hama.com](http://www.hama.com) -> 00131970, 1T131970 -> Downloads.

Frequency band(s)	2402 – 2480 MHz
Maximum radio-frequency power transmitted	7.5 dBm (5.62 mW) EIRP

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Thomson is a trademark of
license to HAMA GmbH & Co KG,